611067
611068

Contains CE-approved medical products.

Sunmatic The Safety Company AB

Sörredsvägen 113 | 418 78 Göteborg | Sweden
+46 31 799 16 00 | info@housegard.com
www.housegard.com

RFX+CARE®

RFX+CARE Manufacturing Co., Ltd.
No. 599 Wuxie Road Yuecheng District Shaoxing,
Zhejiang, China 312000

EU REP
RFX+CARE International A/S
Gammel Strandvej 22B, 2990 Nivå, Denmark

Water resistant plaster

72x19mmx27pcs+72x25mmx18pcs

Textile plaster

72x19mmx27pcs+72x25mmx18pcs

Intended use:

For use on minor cuts and abrasions. Protects minor wounds. Read instructions prior to use; retain for future reference.

Instruction for use:

Clean wound and surrounding area and dry skin before applying the adhesive bandage. Remove adhesive bandage from protective wrapper. Position pad over wound and press firmly on adhesive area to ensure adhesion. Change dressing every 24 hours or if it is wet. Keep dry and out of direct sunlight. Store below 25°C.

Warning:

Use immediately after opening protective wrapper. Wrapper is sealed with natural rubber latex. Do not use on major wounds. If wound bleeds excessively or skin becomes irritated or infected remove adhesive bandage/plaster. Not suitable for children under 2 years old. In rare cases, a reaction to the adhesive bandage/plaster may occur in small children with exceptionally sensitive skin. This may be due to heat, moisture and other factors when plaster is used especially in the face. In both cases consult a doctor immediately. If you are diabetic consult your doctor before use.

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Eye Wash 250ml x 1pcs

Intended use:

Sterile saline solution for emergency rinsing of a wound or flushing the eyes. Sterile Sodium Chloride 0.9%

Instruction for use:

Twist the lid. Rinse the eye, carefully.

Warning:

Not for injection. Not to be taken internally. Do not use if damaged. Once opened, discard any unused solution. Use before the expiry date. Risk of infection by reuse. Storage temperature 5-30°C.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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